WIDEX **UNIQUE™**

Notkunarleiðbeiningar  
**WIDEX UNIQUE™ Fjölskyldan**

U-FS model  
Hátalari-í-eyra

**WIDEX**<sup>®</sup>  
HIGH DEFINITION HEARING

## **Pitt Widex® heyrnartæki**

(Fillist út af heyrnarfræðingi)

Þitt tæki (tegund og númer):

PROGRAM	
<input type="checkbox"/> Universal	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Quiet	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Transport	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Urban	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Party	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Music	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> T	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> M+T	<input type="checkbox"/> Audibility Extender
SPECIAL PROGRAMS	
<input type="checkbox"/> Zen	<input type="checkbox"/> Audibility Extender

<b>PROGRAM</b>	
<input type="checkbox"/> Phone	<input type="checkbox"/> Audibility Extender
<b>SMARTTOGGLE PROGRAMS</b>	
<input type="checkbox"/> Zen+	<input type="checkbox"/> Audibility Extender
<input type="checkbox"/> Phone+	<input type="checkbox"/> Audibility Extender

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### EC directives

#### Directive 1999/5/EC

Hereby, Widex A/S declares that this U-FS is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC.

A copy of the Declaration of Conformity according to 1999/5/EC can be found at:

<http://widex.com/doc>

## Information regarding disposal

Electrical and electronic equipment (EEE) contains materials, components and substances that can be hazardous and present a risk to human health and the environment when waste electrical and electronic equipment (WEEE) is not handled correctly.

Do not dispose of hearing aids, hearing aid accessories and batteries with ordinary household waste. (Continues on next page).

(Continued) Hearing aids, batteries and hearing aid accessories should be disposed of at sites intended for waste electrical and electronic equipment, or given to your hearing care professional for safe disposal. Proper disposal helps to protect human health and the environment.

## **FCC and IC statements**

FCC ID: TTY-UFS

IC: 5676B-UFS

Federal Communications Commission Statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:

(1) This device may not cause harmful interference, and

(2) this device must accept any interference received, including interference that may cause undesired operation.

**NOTE:**

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be deter-

mined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

NOTE:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Changes or modifications to the equipment not expressly approved by Widex could void the user's authority to operate the equipment.

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Under Industry Canada regulations, this radio transmitter may only operate using an antenna of a type and maximum (or lesser) gain approved for the transmitter by Industry Canada.

To reduce potential radio interference to other users, the antenna type and its gain should be so chosen that the equivalent isotropically radiated power (e.i.r.p.) is not more than that necessary for successful communication.

This device complies with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions:

- (1) this device may not cause interference, and
- (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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## Symbols

Symbols commonly used by Widex A/S in medical device labeling (labels/IFU/etc.)


Symbol	Title/Description
	<p><b>Manufacturer</b></p> <p>The product is produced by the manufacturer whose name and address are stated next to the symbol. If appropriate, the date of manufacture may also be stated.</p>
	<p><b>Catalogue number</b></p> <p>The product's catalogue (item) number.</p>

Symbol	Title/Description
	<p data-bbox="270 213 584 242"><b>Consult instructions for use</b></p> <p data-bbox="270 270 866 379">The user instructions contain important cautionary information (warnings/precautions) and must be read before using the product.</p>
	<p data-bbox="270 412 370 441"><b>Warning</b></p> <p data-bbox="270 469 866 534">Text marked with a warning symbol must be read before using the product.</p>

Symbol	Title/Description
	<p><b>WEEE mark</b></p> <p>“Not for general waste”</p> <p>When a product is to be discarded, it must be sent to a designated collection point for recycling and recovering to prevent the risk of harm to the environment or human health as a result of the presence of hazardous substances.</p>
	<p><b>CE mark</b></p> <p>The product is in conformity with the requirements set out in European CE marking directives.</p>

Symbol	Title/Description
	<p><b>RCM mark</b></p> <p>The product complies with electrical safety, EMC and radio spectrum regulatory requirements for products supplied to the Australian or New Zealand market.</p>
	<p><b>Interference</b></p> <p>Electromagnetic interference may occur in the vicinity of the product.</p>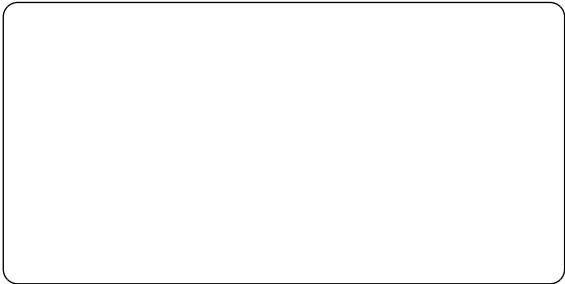

**WIDEX A/S**

Nymoellevej 6, DK-3540 Lyngby, Denmark  
[www.widex.com](http://www.widex.com)

**CE** 0459

**Manual no.:**  
9 514 0317 001 #01  
**Issue:**  
2015-09